Patient information leaflet

Read this package insert carefully in its entirety before using this product

FERTI-LILY

A silicone cup to increase the chance for natural conception/establishment of pregnancy

FERTI-LILY is a cup-shaped medical device, to assist and increase the chance for naturally becoming pregnant. The cup is made of medical grade silicone, intended for insertion into the vagina after sexual intercourse, and is reusable. FERTI-LILY pushes the sperm cells in the direction of the cervix and keeps the sperm cells close to the cervix for a prolonged time. The product thus increases the physical proximity of the sperm cells to the cervix, increases the number of sperm cells that swim across the cervix, and protects them from the acidic environment (low pH) in the vagina. These actions increase the chance for becoming pregnant. The product thus helps to achieve conception naturally.

FERTI-LILY is intended for insertion into the vagina after sexual intercourse. It therefore does not interfere with the spontaneity of sexual intercourse and does not interfere with intimacy. FERTI-LILY is made of soft, flexible silicone, and can be easily inserted into the vagina. FERTI-LILY can remain in the vagina for up to an hour after insertion. It thus increases the amount of time that the sperm cells are present in the vicinity of the uterine cervix.

The product is composed of the cup itself, to which is attached an elongated stem-like part (below, this will be called the "stem"), which helps with the removal of the product from the vagina (Figure 1).

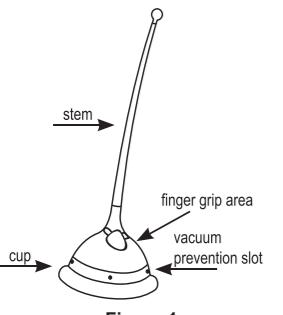
Figure 1

- People with long fingernails - should be very careful when inserting and removing the product.

• Use FERTI-LILY during the ovulation period. Methods of identifying ovulation may be used in order to assess which days are the most fertile during the monthly cycle.

Step one:

Before each use, rinse the product in hot water for 15-20 seconds (the temperature of the water should be like the temperature of bath water - approximately 40°C). Remember to put the FERTI-LILY close to your bed, so that you will not need to get out of bed after having sexual intercourse.

Step two:

After completion of sexual intercourse, after your partner has ejaculated, bring the FERTI-LILY to yourself. Note that you should not get up or stand before inserting the product in your vagina, in order to avoid the flow of semen out of your vagina. Insertion of the product is done while lying down, with one leg drawn towards the chest.

1. Turn the cup inside out so that it looks like an umbrella (Figure 2).
2. Hold the FERTI-LILY so that the stem points towards your legs, with one finger inside the "umbrella" (Figure 3).

Figure 2

Figure 3

Step three:

1. Gently and very carefully, insert the FERTI-LILY cup into your vagina using the finger that is inside the cup (Figure 4). Insertion of the cup will push the sperm towards the opening of the cervix. If you have difficulty inserting the cup, it can be folded in order to reduce its size while it is being inserted. The tip of the stem will usually remain outside the vagina. For some women, the entire stem may enter the vagina. This is completely normal.

2. When the FERTI-LILY cup reaches the end of the vaginal canal, pull the stem gently. This action will cause the cup to resume its original cup shape (Figure 5).

How do you know whether FERTI-LILY has resumed its cup shape? The two shapes can be distinguished by the sensation that they create in the vagina - when the product is in its cup shape, its rim can be felt on the vaginal walls. On the other hand, when the product is in an "umbrella" shape, it can barely be felt on the vaginal walls (the difference in feeling can be simulated by holding the product in your hand). Even if you are unable to distinguish whether the product is cup shaped or umbrella shaped, don't worry - even when the product is umbrella shaped, it prevents sperm cells from flowing out of your vagina, and it may be left like that in your vagina. The ability to correctly insert the product will improve with use.

Leave the FERTI-LILY in your vagina for at least 20 minutes (Figure 6). The product may remain in your vagina for one hour. While the product is in your vagina, you may get up, bathe, relieve yourself (urine and feces) and go about your daily activities without concern that the sperm cells may flow out of your vagina. When FERTI-LILY is in your vagina, there is no need to lie down with your legs up.

Figure 4

Figure 5

Figure 6

When should this product not be used?

Do not use in case of known sensitivity to the product's ingredients. Do not use the product if there is vaginal infection of yeast origin or of bacterial origin. Use of the product can be resumed after recovery from the infection. However, if you used FERTI-LILY at the time of the infection, it is recommended to replace the FERTI-LILY with a new one in order to ensure that the infection does not return. Do not use at the time of your menstrual period.

In the following instances, a physician should be consulted prior to use:

If you have suffered in the past from **Toxic Shock Syndrome** (a rare syndrome that is often connected to the use of tampons), if you have a prolapsed or reversed uterus or any other structural problem that could interfere with the insertion or removal of the product, or if you are after childbirth, miscarriage or abortion - consult with a physician prior to using FERTI-LILY.

Side effects:

On first use, you may experience mild cramps. These sensations pass rapidly as you get used to the product.

Side effects that require special attention:

If you experience any of the following symptoms that could indicate toxic shock syndrome, remove the product and consult a physician immediately: vomiting, diarrhea, fever, headaches, rash, muscle pain, confusion and/or convulsions.

In any instance in which you experience side effects not specified in this leaflet, or if there is a change in your general health, stop using the product, and if needed consult a physician.

Directions for Use:

- Before using the product, read the information in this leaflet through to the end.
- Keep this leaflet for future use.
- This leaflet does not replace medical advice. If you suffer from a medical problem or a suspected medical problem, consult with a physician before using the product.
- To see a video with instructions for use of the product, visit the following website: www.FERTI-LILY.com
- **Before first use, boil the product in an open pot of water for 5-10 minutes.** Then dry the product and store it in the enclosed cotton bag.
- For first time users – before having sexual intercourse, it is recommended to try to insert the product, place it in the vagina and remove a few times, in order to make sure that you are familiar with the insertion and removal procedure. The first times may be challenging, but with use, getting used to the product will rapidly make the process easier. While practicing, you may use a water-based lubricant. After that, wash the FERTI-LILY in accordance with the cleaning instructions that appear below.

Figure 7

Figure 8

How should I clean the FERTI-LILY?

Clean the product with warm water only! (The temperature of the water should be like the temperature of bath water - approximately 40°C). **Do not use soaps or other cleaning agents**, including the following substances: vinegar, tea tree oil, scented soap, soap with oils (such as olive oil or peppermint oil), oil-based soap, alcohol, antibacterial soap, hand sanitizer, pre-moistened wipes, dishwashing liquid, hydrogen peroxide, bleach or any other chemical substance. These substances could damage the silicone and cause vaginal irritation.

Step one:

Before each use, rinse the product in hot water for 15-20 seconds

(the temperature of the water should be like the temperature of bath water - approximately 40°C).

Remember to put the FERTI-LILY close to your bed, so that you will not need to get out of bed after having sexual intercourse.

Step two:

After completion of sexual intercourse, after your partner has ejaculated, bring the FERTI-LILY to yourself. Note that you should not get up or stand before inserting the product in your vagina, in order to avoid the flow of semen out of your vagina. Insertion of the product is done while lying down, with one leg drawn towards the chest.

1. Turn the cup inside out so that it looks like an umbrella (Figure 2).

2. Hold the FERTI-LILY so that the stem points towards your legs, with one finger inside the "umbrella" (Figure 3).

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